



ComPAS Install & Update Instructions on Citrix

No.	INSTRUCTIONS for INSTALLING ComPAS on CITRIX
1	Install ComPAS using provided installer, accepting defaults when prompted Important: Right-click the ComPAS installer icon and "run as administrator" to install
2	Add the following programs to the Server DEP exclusion list: C:\Program Files (x86)\Morgan Scientific\ComPAS\ComPAS.exe C:\Program Files (x86)\Morgan Scientific\ComPAS\ComPAS_TaskManager.exe
3	Right-click ComPAS icon and select "Run As Administrator" (Note: This is only required on first run to authenticate to the ComPAS database)
4	You will be prompted to enter Database Server details; please select the following responses: a) Select "Remote Database" radio button b) For Computer, enter: <enter name of SQL Server without quotes or backslashes> c) Select "Named Instance" radio button d) In text input box below "Named Instance", enter: None (Important: this is case sensitive) e) For Catalog, enter: ComPAS
5	You will be prompted to "Authorize" this new workstation. Select "Administrator" as the username.; password is pulmonary.
6	Publish ComPAS TaskManager on Citrix: When publishing, please use "ComPAS TaskManager" program found at: "C:\Program Files (x86)\Morgan Scientific\ComPAS\ComPAS_TaskManager.exe" Please name it "ComPAS TaskManager" and please do not restrict the end-user's maximum resolution. ComPAS is capable of printing patient data and exporting to a disk. Depending on hospital policy, it is up to you to allow such operations to work against the end-user's local computer, which in some cases may be offsite.
	TIP: If "Provider cannot be found" appears when launching ComPAS for the first time, here is the solution: o Download and install Microsoft SQL 2008 R2 Native Client x64 or from this location: http://files.morgansci.com/Public/ITSPUT/SQL/sqlncli.msi Please contact Morgan IT Support for access to above link.

No.	INSTRUCTIONS for UPDATING ComPAS on CITRIX
1	Follow step 1 above only to update ComPAS on Citrix