



ComPAS Software Quick Update Instructions

Rev: 20170731.01

1. Backup Database

2. Login to computer as a local Administrator
3. Right click ComPAS_XXXX.exe and Run as Administrator – follow along with defaults
4. Right click ComPAS icon and Run as Administrator
 - a. First Workstation will update the ComPAS database (Database updates only once)
5. Verify Compatibility on Workstations:
 - a. Right click ComPAS icon on desktop and select Properties
 - b. Navigate to Compatibility tab
 - c. Click button at the bottom to 'Change for all users'
 - d. Select Compatibility Mode – Windows XP Service Pack 3
 - e. Checkmark for – 'Disable display scaling on high DPI settings'
 - f. Click Apply and Ok to exit

** Any issues or questions, please call Morgan Scientific IT, Thomas Fallon, (800) 525-5002.

** Please download and run ComPAS Quick Support on the workstation and have the 9-digit code ready for support.

http://files.morgansci.com/Public/Software/ComPAS_QS.exe