



ComPAS Software First-time Install Instructions

Rev: 20170330.01

1. Login to computer as a local Administrator
2. Right click ComPAS_SQLDEPLOY_v2.exe, Run as Administrator [Client Standalone Database Only]
3. Right click ComPAS_XXXX.exe, Run as Administrator – follow along with defaults
4. Compatibility Settings - Right click ComPAS icon on desktop and select Properties
 - 1) Click Compatibility tab
 - 2) Click “Change settings for all users” located at bottom
 - 3) Select “Run this program in compatibility mode for Windows XP Service Pack 3”
 - 4) Select ‘Disable display scaling on high DPI settings’
 - 5) Click OK twice to exit
5. Right click ComPAS icon and Run as Administrator
 - a. First-time setup installer will initiate [May take a minute for the installer to launch]
 - b. Click Next to select everything
 - c. At the window to select local vs. remote database, leave the defaults for a local database
 - d. The installer will run for approximately 10 minutes
6. Upon completion, Right-click ComPAS icon on desktop and Run as Administrator to launch
 - a. For first-time installs with PFT equipment, please call Morgan Scientific Product Support to assist with setting up the PFT equipment.

** Any issues or questions, please call Morgan Scientific IT, Thomas Fallon, (800) 525-5002.

** Please download and run ComPAS Quick Support on the workstation and have the 9-digit code ready for support.

http://files.morgansci.com/Public/Software/ComPAS_QS.exe